



# ConFirm®

## Mail-in Sterilizer Monitoring Service

Test Results in  
**24 HOURS...**  
**CONFIRMED!**

### Test Results In 24 hours

Steam cycle test results in 24 hours. Test failures are immediately phoned to your office. Complete sterilization test results are mailed, faxed or e-mailed quarterly.

### Check Results Online

Your results are always available online with our secure, confidential web site.

### FREE Reminder Program

ConFirm's FREE reminder program helps you keep on your regular testing schedule.

### Peace of Mind

Third party documentation of test results provides a tangible record as well as a legal document where proof of sterility testing must be provided.

### Easy to Use

**1. Sterilize:** Simply place the bacterial spore test strip(s) in a loaded sterilization chamber and process. **2. Mail:** Return strips to envelope and mail. Postage is prepaid.\* **3. Get Results:** Test failures are immediately phoned to your office. Results are also available online. **4. Get Proof:** ConFirm Monitoring Systems provides third party documentation regarding the effectiveness of your sterilizer, which provides added confidence.

\*Postage not prepaid for Value Test.

### Economical

ConFirm is one of the most cost-effective methods of biological monitoring.

### Sterility Assurance

ConFirm mail-in uses highly resistant bacterial spores to provide the most reliable method of testing for an effective sterilization cycle.

### Universal

The ConFirm strips contain both *Bacillus stearothermophilus* and *Bacillus subtilis* spores and therefore may be used to test all sterilization processes – steam, chemical vapor, dry heat and ethylene oxide.

 **ConFirm®**  
Monitoring Systems, Inc.


109 Inverness Drive East, Unit F  
Englewood, CO 80112-5105  
Toll Free: 1-800-819-3336  
[www.confirmonitoring.com](http://www.confirmonitoring.com)

# ConFirm® With Confidence



## Are You Meeting Regulations?

Sterilization Regulations for Dental Practices

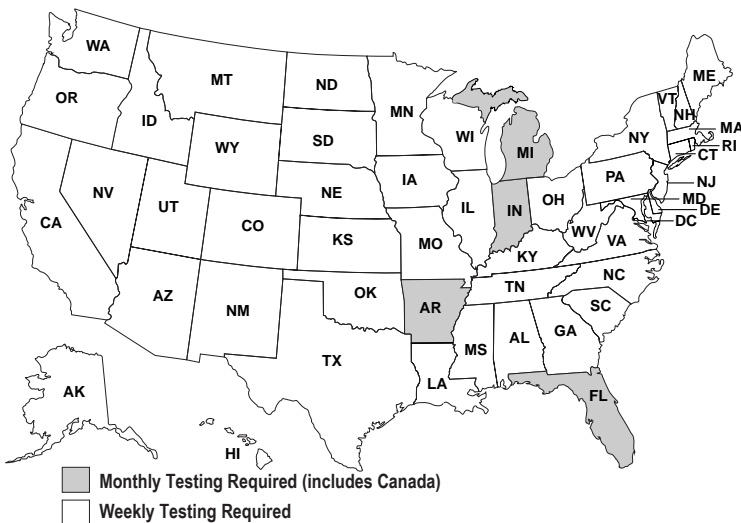

## Organization Guidelines

- |             |   |
|-------------|---|
| <b>CDC</b>  | <ul style="list-style-type: none"> <li>Weekly</li> <li>Each load of implantables</li> <li>Each EtO cycle</li> </ul>                       |
| <b>OSHA</b> | <ul style="list-style-type: none"> <li>Regularly. Follows CDC recommendations</li> </ul>  |
| <b>ADA</b>  | <ul style="list-style-type: none"> <li>Weekly</li> <li>Each load of implantables</li> </ul>   |
| <b>OSAP</b> | <ul style="list-style-type: none"> <li>Weekly</li> </ul>  |
| <b>VA</b>   | <ul style="list-style-type: none"> <li>Daily for steam and dry heat</li> <li>Each load of implantables</li> <li>Each EtO cycle</li> </ul> |
| <b>AAMI</b> | <ul style="list-style-type: none"> <li>Weekly</li> <li>Each load of implantables</li> <li>Each EtO cycle</li> </ul>                       |
| <b>AORN</b> | <ul style="list-style-type: none"> <li>Weekly</li> <li>Each load of implantables</li> <li>Each EtO cycle</li> </ul>                       |

To order call:

**1-800-819-3336**

**ConFirm®**  
Monitoring Systems, Inc.

109 Inverness Drive East, Unit F  
Englewood, CO 80112-5105  
Fax: 303-699-8255

## Order Information

| Description   | Order No. |
|---|-----------|
| <b>ConFirm Mail-In Test</b>   |           |
| 3 bacterial spore test strips (2 tests/1 control), postage prepaid.     |           |
| 6 Tests   | CST060    |
| 12 Tests  | CST120    |
| 48 Tests  | CST480    |
| <b>ConFirm Value Test</b>   |           |
| 2 bacterial spore test strips (1 test/1 control), postage not included. |           |
| 12 Tests  | CVT120    |
| 52 Tests  | CVT520    |